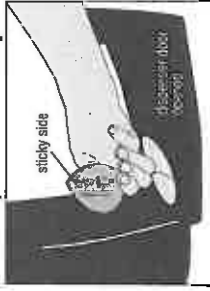
# EZ HAND HYGIENE “NO TOUCH” DISPENSERS

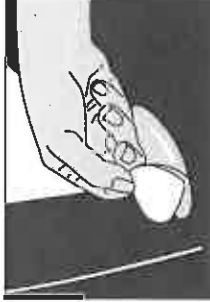
Instructions for Applying Dispenser Label / Directives pour la pose de l'étiquette de la distributrice / Instrucciones para Aplicar la Etiqueta del Dispensador

1. Peel label from backing. The adhesive is on the FRONT of the label. 1. Retirer le verso de l'étiquette. Le côté adhésif est le RECTO de l'étiquette. 1. Pele la etiqueta del refuerzo. El lado con adhesivo pegajoso es la parte DELANTERA de la etiqueta.

2. Open dispenser. Place label sticky side down on INSIDE of clear oval window. 2. Ouvrir la distributrice. Poser l'étiquette, le côté adhésif vers le bas, sur L'INTÉRIEUR de la fenêtre ovale transparente. 2. Abra el dispensador. Coloque la etiqueta, el lado pegajoso hacia abajo, sobre el INTERIOR de la ventana oval transparente.

3. Close front of dispenser. 3. Fermer la partie avant de la distributrice. 3. Cierre la partie delantera del dispensador.

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
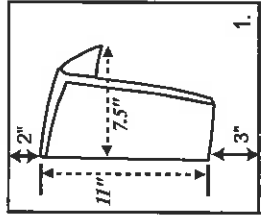
For most applications, tape mounting is adequate. For porous, textured or wallpapered surfaces, screw mounting or a combination of screw and tape mounting is recommended.

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
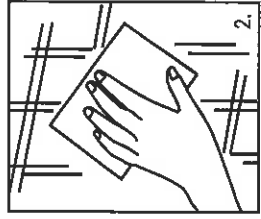
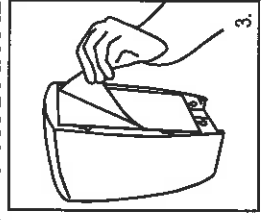
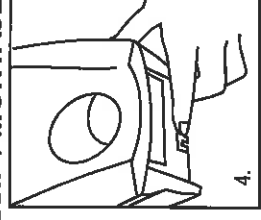
4. Open the dispenser by pushing up on the release latch. The latch is on the bottom back edge of the dispenser in the center. NOTE: See "Keyed Locking Option" if you wish to convert your dispenser to keyed system. While pushing the release latch, open the dispenser by placing your other hand on the top and guiding the front cover forward. Apply pressure from inside the dispenser over the taped area.

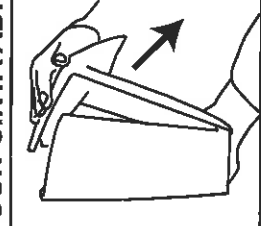NOTE: Allow dispenser to remain empty for a minimum of eight hours before loading.

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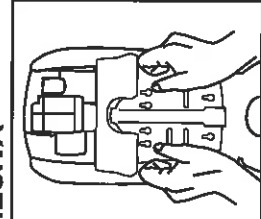
## SCREW MOUNTING / MONTAGE À VIS / MONTAJE CON TORNILLOS

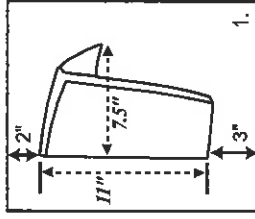1. **IMPORTANT:** Dispenser opens to the front. For ideal placement, the dispenser should be mounted 3" above any countertop, fixture or obstruction and 2" below any upper cabinet, fixture or obstruction.

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

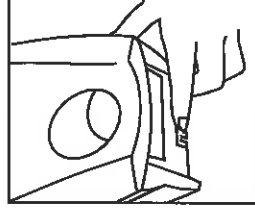
3. Open the dispenser by pushing up on the release latch. The latch is on the bottom back edge of the dispenser in the center. NOTE: See "Keyed Locking Option" if you wish to convert your dispenser to keyed system. While pushing the release latch, open the dispenser by placing your other hand on the top and guiding the front cover forward.

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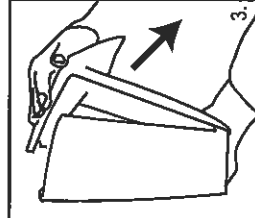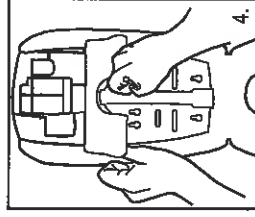4. Hold the dispenser against the wall in the proper position and mark the appropriate slots. Drill holes in marked slots. Attach dispenser to the wall with the wall anchor and screws provided. You may choose to remove a portion of the protective backing from the tape to help hold the dispenser in position as you complete the installation.

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
## CHANGING THE BATTERIES. 6-AA BATTERIES. NO SCREWDRIVER NEEDED.

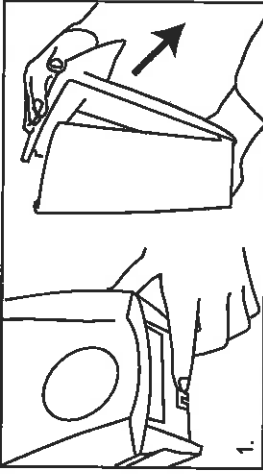Batteries need changing when sensor light  flashes RED.

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
1. Open the dispenser by pushing up on the release latch (or with the key if you have converted to the dispenser to the keyed locking mode). The latch is on the bottom back edge of the dispenser in the center.

While pushing the release latch, open the dispenser by placing your other hand on the TOP, and guiding the front cover forward.

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
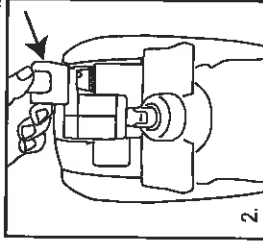
2. Flip up cover of battery compartment in upper right.

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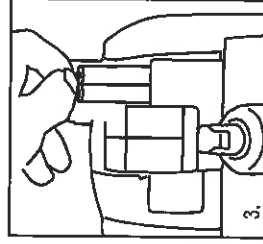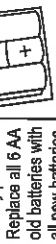
3. Unhook battery terminals and remove battery pack. Replace all 6 AA batteries with all new batteries.

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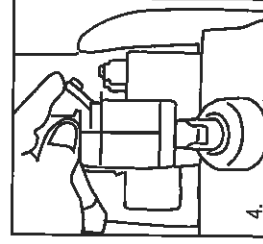
4. Return battery pack to space, attach battery terminals.

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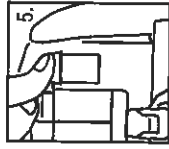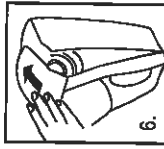
Product loading instructions are on **INSIDE** of dispenser cover and on product bag labels.

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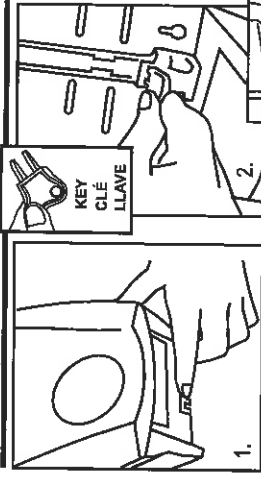
## INSTRUCTIONS FOR "KEYED" LOCKING OPTION


For added security, your new foam dispensing system has a KEYED locking option. NOTE: The dispenser has been shipped to you in the manual, NO KEY REQUIRED mode. If you wish to convert your dispenser to the KEYED locking mode, please follow the instructions. The conversion can be completed before or after dispenser installation.

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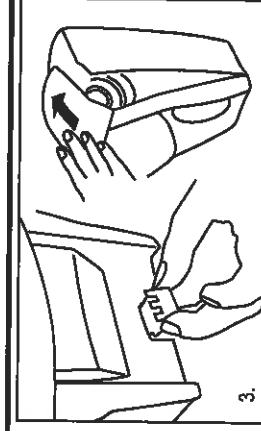
1. Open the dispenser by pushing up on the release latch. The latch is on the bottom back edge of the dispenser in the center. While pushing the release latch, open the dispenser by placing your other hand on the top and guiding the front cover forward.

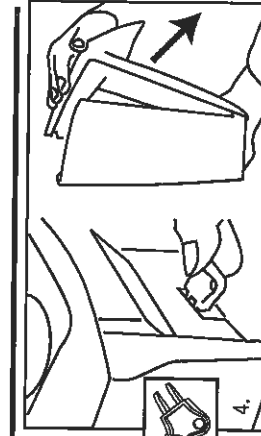2. From **INSIDE** the dispenser, insert the key into the back of the dispenser in the key slot and push until the latch piece is ejected from the bottom of the dispenser.

3. Re-install the latch piece in the bottom of the dispenser with the **SLOTTED SIDE AWAY** from the wall and toward the front of the dispenser. Push the latch piece into the dispenser until it snaps into place. Close the dispenser by placing your hand on **TOP** of the dispenser, or the front of the dispenser below the push pad, and pushing up on the cover until it is latched into place. **NOTE: Do not attempt to close dispenser by pushing on push pad**

4. To **OPEN**, insert the key into the slots of the release latch on the bottom of the dispenser and push the key up. Place your other hand on top of the dispenser and guide the front cover forward. **IMPORTANT: The dispenser can be converted back to the NO KEY REQUIRED mode. Complete steps #2, and #3, and insert the latch piece with the SLOTTED SIDE TOWARD the wall.**

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